
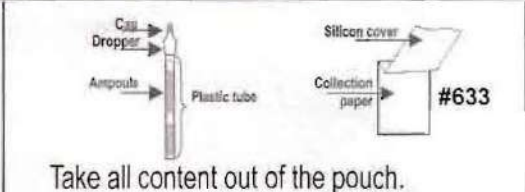

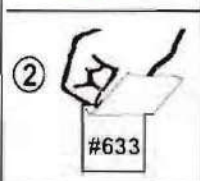
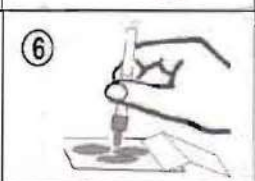
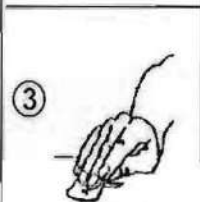
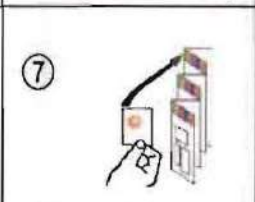
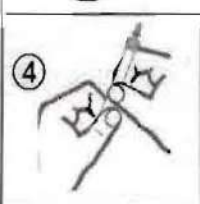
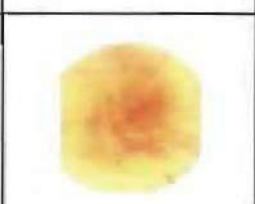



PEN TEST-CANNABIS TEST

Please read instruction carefully before use.

| | | | |
|--|--|---|---|
|  |  <p>Take all content out of the pouch.</p> |  | <p>Hold the cap with your finger and shake well for 10 seconds.</p> |
|  | <p>Take the (#633) collection paper. Peel back the protective silicon cover. Hold it at the edge. Do not touch the exposed surface to avoid pre-contamination.</p> |  | <p>Remove cap, pointing the tube away from your body. Point the tube down and apply few drops on the paper to cover sampling area.</p> |
|  | <p>Wipe the suspected area, or touch the suspected substance with the test paper.</p> |  | <p>Observe for an immediate appearance of reddish orange stains, indicating the presence of Cannabis (Hashish, and Marijuana) check carefully with the color chart. Discard the test ingredients.</p> |
|  | <p>Holding the tube upright, squeeze the plastic tube to break the ampoule. Do not squeeze tube again over broken glass.</p> |  | <p>Cannabis (Hashish, Marijuana)</p>  |
| <p>Only an immediate color reaction should be considered as positive. Only colors similar to the color chart should be considered as positive results. For each test a new collection paper must be used. Use the collection paper enclosed in each pouch. Caution: keep out of reach of children. In case of eye or skin contact, flush well with water.</p> | | | |

1. This pouch contains two tests. Once ampoule is broken, the test should be used immediately.
2. Cannabis Test detects the presence of: Cannabis (Hashish, Marijuana).
3. Use color chart to interpret the results.