







	Cannabispray	Herosol	Coca-Test	Meth Test
	Slide one collection paper from the collection paper dispenser #530	Slide one collection paper from the collection paper dispenser #530	Slide one collection paper from the collection paper dispenser #530	Slide one collection paper from the collection paper dispenser #630
	Peel the protective silicon layer, exposing the sticky collection surface.	Peel the protective silicon layer, exposing the sticky collection surface.	Peel the protective silicon layer, exposing the sticky collection surface.	Peel the protective silicon layer, exposing the sticky collection surface.
	Wipe the suspected area or touch the sticky side of the collection paper to the suspected substance	Wipe the suspected area or touch the sticky side of the collection paper to the suspected substance	Wipe the suspected area or touch the sticky side of the collection paper to the suspected substance	Wipe the suspected area or touch the sticky side of the collection paper to the suspected substance
	Spray the paper briefly with Cannabispray-1 can from a distance of 15 cm (6").	Spray the paper briefly with the Herosol spray can from a distance of 15 cm (6").	Spray the paper briefly with the Coca-Test spray can from a distance of 15 cm (6").	Spray the paper briefly with Meth Test-1 can from a distance of 15 cm (6").
	Spray with Cannabispray-2 can onto the same area (on the collection paper) previously sprayed by Cannabispray-1.			Spray with Meth Test-2 can onto the same area (on the collection paper) previously sprayed by Meth Test-1.
	Observe for an immediate appearance of reddish brown stains indicating the presence of Hashish or Marijuana.	Observe for an immediate appearance of Dark Violet stains indicating the presence of Heroin, LSD, Methadone, Methamphetamines and/or Amphetamines.	Observe for an immediate appearance of Turquoise stains indicating the presence of Cocaine, Crack, Buprenorphine or related drugs.	Observe for Dark Blue stains that will appear within 30 seconds, indicating the presence of Methamphetamines
<p>SAFETY PRECAUTIONS: Use in well ventilated area. Spray away from your body. Do not inhale. Avoid contact with skin. In case of eyes or skin contact, flush with plenty of water. FLAMMABLE. Do not use near fire. Warning: Contents under pressure. Do not puncture or incinerate. Do not store at temperature exceeding 50°C (120°F)</p>				