


CheckMate Infidelity Test Kit

1. Wet the suspect stain with 2 to 3 drops of bottled water
2. Collect the stain by swabbing wetted area with Q-Tip
3. Cut the ends of Q-Tip off and place in the provided vial
4. Use provided dropper to place 20 drops of water into vial
5. Close vial and shake for 60 seconds
6. Use dropper to extract small amount of mixed liquid from vial
7. Place 4 drops of liquid into round hole on test cassette
8. Read results at exactly 10 minutes
9. Semen is indicated by line under the T and line under the C
10. Negative test for semen is indicated by 1 line under the C

Find out what's really going on with CheckMate
Call 800 498 5015 for more information

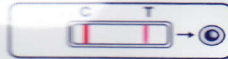

Positive



Negative

1 Test

