

The CheckMate® 5 Minute Semen Detection Test Kit

Instructions: Read Completely Before Use

*CheckMate® test for the presence of semen, but will not identify the person the semen came from. If you suspect your partner of having sex, with someone other than yourself, you must not have **sexual intercourse, with your partner, for at least 7 days.** Semen can be detected for up to 84 hours after showering.*

Directions

Step 1: Use the supplied dropper and remove a small amount of liquid from the bottle and wet the suspected stain with 2-3 drops. Allow the liquid to soak into the fabric and proceed to step two before fabric has dried.

Step 2: Using one of the supplied blotting papers, blot the entire wetted stain area, pushing down gently in different places, until the blotting paper is thoroughly wet. This transfers some of the stain from the fabric tested to the blotting paper. Set the blotting paper aside to dry. Note: Blotting Paper must be **100% dry** prior to proceeding to step three.

Step 3: Mixing the solution: Using the supplied dropper, remove 10 ml of the liquid (2 dropper fills) from the liquid bottle and empty liquid (from dropper) into the bottle with power reagent. Shake the mixed solution for at least 30 seconds to mix completely.

Step 4: Use the supplied dropper and remove a small amount of the mixed solution (from step 3) and wet the blotting paper (from step 2) with 3 drops (of the mixed solution). Once you drop the mixed liquid onto the blotting paper start timing. **The blotting paper will turn PURPLE almost immediately, but definitely within 15 seconds If semen is present.** If a PURPLE color (light or dark) does not appear within 15 seconds, the stain tested is NOT semen. Note: If the blotting paper turns Purple after 15 seconds or the blotting paper turns any other color before or after 15 seconds, the tested stain is **NOT** semen. Store the unused mixed solution in a cool dry place and use within **24 hours**...solution should be discarded after 24 hours.

Note: By following steps 1-4, no traces of the test will be left behind on the fabric being tested. If you are not concerned about staining the fabric containing the suspected semen, and the fabric is a light color (like white), you can place the mixed solution directly onto the stained fabric. If semen is present, you will see a purple color appear almost immediately, but definitely within 15 seconds.

If you have questions call one of our customer service specialists at 1-800-498-5015 9:00 AM to 5:00 PM EST or email info@getcheckmate.com

Warning: Wash hands thoroughly after using this product. Use this product with adequate ventilation and keep away from children. Harmful if swallowed, inhaled or absorbed through the skin. If swallowed, give victim large amount of water and induce vomiting...then seek medical attention. Because of the nature of this product, and the conditions and circumstances under which it may possibly be used, the makers and distributors assume no responsibility whatsoever for actions that may arise as a result of using this product. CheckMate is a screening test and not intended to be used as legal evidence in court of law. By using this product, the user agrees to this terms and conditions.