

Cocaine Trace Wipes

Directions:

1. Before use, put on the safety gloves provided
2. Remove Cocaine wipe from wrapper and wipe surface
 - a. Note: If surface is excessively dirty or contains colored matter, results may be inconclusive.
3. If Cocaine, Crack/Freebase is present, wipe will indicate light blue
4. After use, clean surface tested with damp cloth and wash hands with soap and water
5. Safety gloves and used wipes can be disposed of with normal waste

NOTE:

- a. Sensitivity of the test is < 0.02 mg depending on the purity of the material tested
- b. Not to be used on the human body as body cleaning wipes.
- c. Store in a cool, dry place.